



# 5% Sodium Fluoride Varnish

— Your partner in cavity prevention



### Packaging

- 10g / Tube
- 0.4g\*50 / Box

## Pro Shield Fluor Protector

To keep teeth healthy, small amounts of fluoride are needed during all stages of tooth growth. Several dental organizations and research reports recommend the professional use of fluoride varnish. Pro Shield is used for the prevention of recurrent cavities, cavity development, decalcification around orthodontic appliances, as well as pit and fissure (occlusal) cavities.

Based on our patented technology, we come up with a unique packaging. The design of separating the medicament and the brush head maximizes the doctor's operational convenience.

### Summary of advantages

- Stable and long-lasting fluorine release — **22,600 ppm fluoride**
- Excellent consistency and long-term performance stability
- Fine and uniform particles enable a stable suspension of sodium fluoride, enhances fluorine release, as well as distribution
- Pleasant banana flavor to improve patients experience
- Unique packaging design facilitates clinical operation
- Light yellow provides visual control

Material color: Light Yellow 



### VINCISMILE GROUP LLC

Add: 20524 Carrey Rd. Walnut, CA 91789, USA

Email: [marketing@vincismile.com](mailto:marketing@vincismile.com)


Shop with us online at [store.vincismile.com](http://store.vincismile.com)

Tel: +1 626 283 5808 Web: [www.vincismile.com](http://www.vincismile.com)



Follow @VinciSmile

## Excellent Operation Experience with Pro Shield

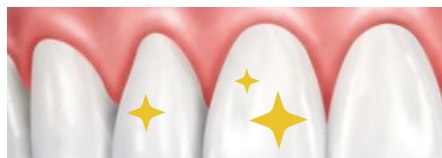


1. Make sure the areas to be treated are clean.



2. Remove excess saliva with triple syringe or cotton pellets.



3. For 10g tube: Squeeze an appropriate amount of material from the tube.

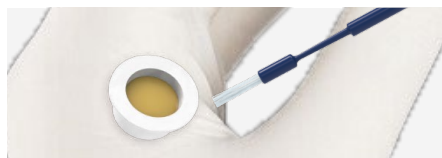

4. For 0.4g single dose: Apply the single dose directly.





5. Use the microbrush to apply a thin layer to the tooth surface.



6. Pro Shield forms a thin film on the tooth surface.

## High Fluoride Release Capability Suitable for Kids and Adults

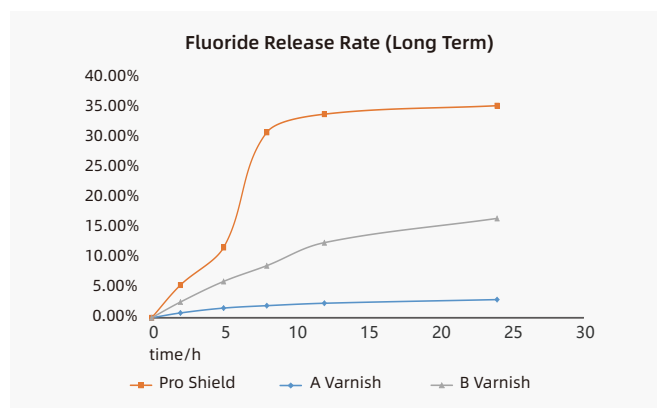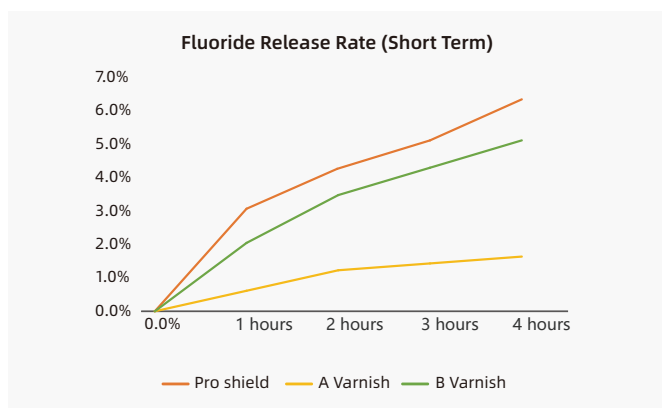

\*Data from VinciSmile Lab

### Clinical application suggestions:

#### 1. Recommended dosage:

Deciduous dentition  $\leq 0.25g$  (5.65mg fluoride)  
 Mixed dentition  $\leq 0.4g$  (9.04mg fluoride)  
 Permanent dentition  $\leq 0.75g$  (16.95mg fluoride)

#### 2. Frequency of use:

Use once every **3-6 months**.

#### 3. Precautions after use:

After using this product, do not brush the teeth, chew food, or drink hot drinks and alcoholic products (beverages, mouthwash, etc.) within **4 hours**.

